

**BREAK FREE
FROM GLASSES
AND CONTACTS!**

EVO ICL™
Evolution in Visual Freedom




Benefits of EVO ICL

.....| **EXCEPTIONAL VISION DAY AND NIGHT**



Experience what life has to
offer throughout the day
**without compromising your
night vision.**^{1,3}

Day or night—you can rely on
EVO for exceptional vision.^{1,3}

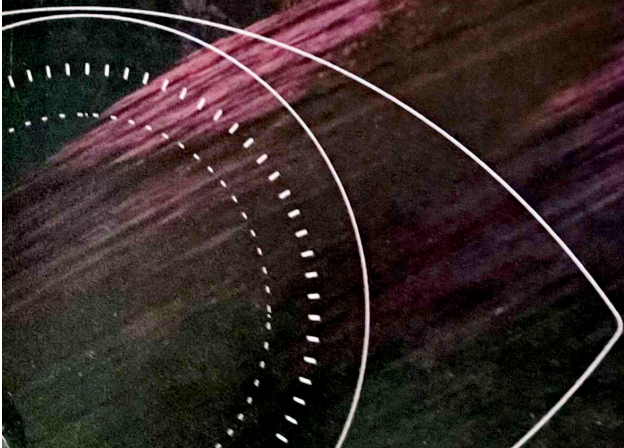

.....| **IN HARMONY WITH YOUR NATURAL EYE**



Similar to adding a lens to a camera, EVO is additive and works together with the natural parts of your eye to improve your vision.

.....| **NO DRY EYE SYNDROME**



Clinically proven to not cause or contribute to dry eye syndrome.⁴

.....| **YOUR FUTURE'S BRIGHT, AND PROTECTED**



EVO is made of Collamer®, a material shown to be safe in the eye. Collamer provides UV protection and contains collagen, a naturally occurring protein in the body.

Created for your well-being


MADE WITH COLLAMER — A SAFE, BIOCOMPATIBLE AND PREMIUM MATERIAL

Collamer is biocompatible with your eye, and is made with collagen, a **protein which naturally occurs in your body**. Shown to be safe and effective, and produced with high quality materials, Collamer is used exclusively by STAAR Surgical in EVO.

SOFT AND FLEXIBLE




EVO is ideal for your eye because of its **soft and flexible structure**. You won't even know it's there, apart from the fact that you have great vision.

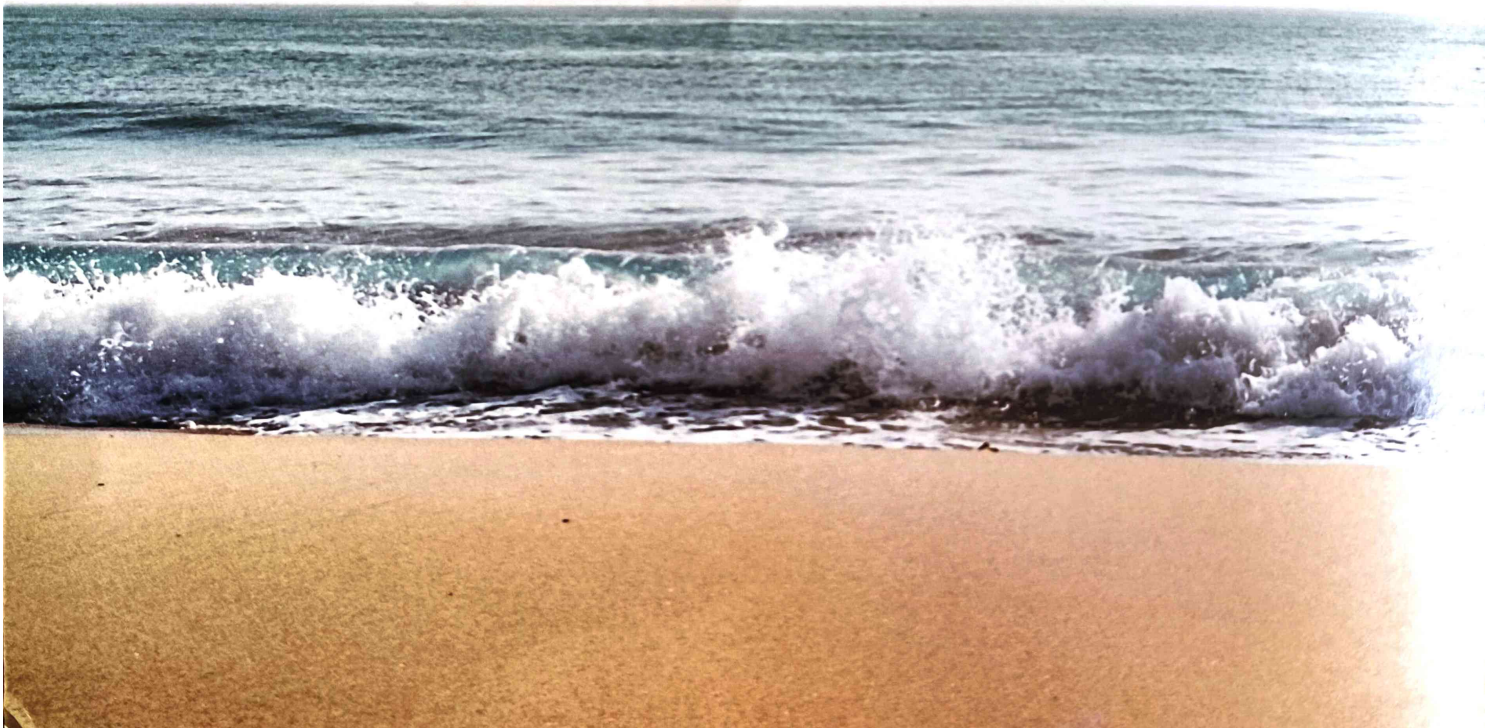

Keep your peace of mind

A REMOVABLE OPTION THAT GIVES YOU FLEXIBILITY FOR THE FUTURE

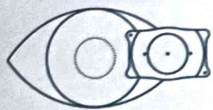


While EVO is designed to permanently correct your vision, **the lens can be removed** by a doctor, if necessary, or to keep pace with advancing technology and your future vision needs.



How ICL works


The STAAR Surgical implantable lens corrects common vision problems, like nearsightedness, astigmatism, and farsightedness. **ICL is placed behind the iris (the colored part of the eye) and right in front of the natural lens**, helping you to achieve sharp, clear vision.¹

NEARSIGHTEDNESS (MYOPIA)

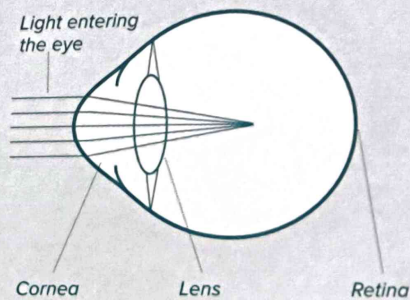


Light rays focus in front of the retina

FARSIGHTEDNESS (HYPEROPIA)*

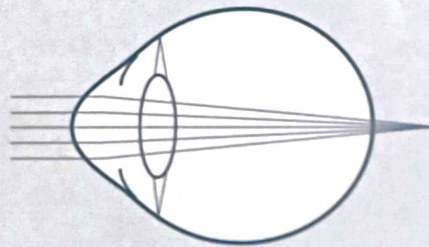


Light rays focus behind the retina

ASTIGMATISM

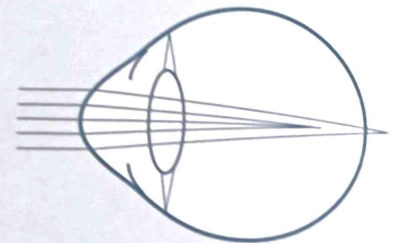


Light rays fail to focus at a single point on the retina

AFTER TREATMENT

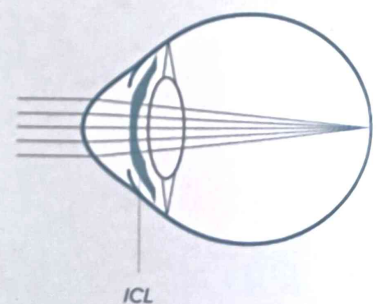

ICL focuses light onto the retina, creating clear vision

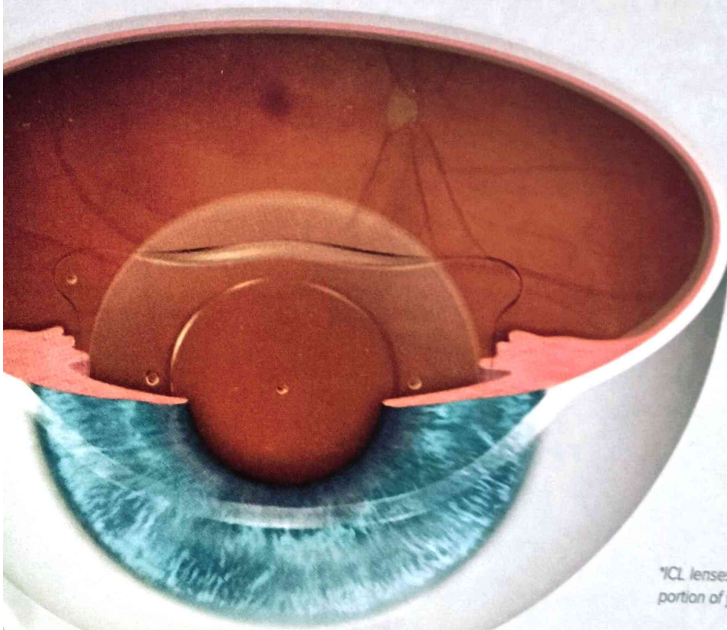

*ICL lenses intended to treat farsightedness are not EVO and require two additional tiny openings in the colored portion of your eyes to ensure there is proper fluid flow after the ICL is implanted.

Quick and Safe Procedure

The EVO procedure is as quick or faster than watching an episode of your favorite binge-worthy series and has a short recovery time.

Getting ready for visual freedom!


Before the day of your EVO procedure, your doctor will perform a series of standard tests to measure your eye's unique characteristics.

Time for your EVO lens!

When you arrive for your procedure, your doctor will administer eye drops to dilate and numb your eye.

THE PROCEDURE FOLLOWS THREE EASY STEPS*:

1

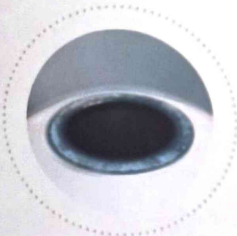

Your doctor will create a small opening at the edge of your cornea to insert **EVO**.

2



EVO is folded and inserted through this small opening.

3



Once the lens is inserted, your doctor will make any necessary adjustments to ensure proper positioning in the eye.

EVO is not visible to you or others.

The procedure is complete!

Many patients notice improved vision nearly immediately.

After-care: Consult with your doctor for further information on the recovery process.

*ICL lenses intended to treat farsightedness are not EVO and require two additional tiny openings in the colored portion of your eyes to ensure there is proper fluid flow after the ICL is implanted.